

Instructions for use Non-health risk use alternative masks, Category 1 general public Alternative masks strictly made for the health crisis linked to COVID 19

Area of use: This mask is intended for the professional environment and is exclusively reserved for Non-health risk uses. It is intended to prevent droplet projections. The use of this mask is part of the strict application of measures related to lockdown, measures of work organization and barrier gestures. This individual mask is intended to be offered to professionals who meet a large number of people during their activities (cashier hostesses, law enforcement officials). It is not intended to be used by caregivers in contact with patients.

Use : Masks have to be used for **4 hours maximum**.

Recommendations for masks handling : Before putting on a mask, wash your hands with soap and water or with an hydro-alcoholic gel

- Apply the mask so as to cover the nose and mouth and make sure to adjust it at best on the face
- Check that there is no air jet in the eyes during a strong exhalation
- When wearing a mask: avoid touching it and do not move the mask. Whenever you touch a used mask, wash your hands with soap and water or with a hydroalcoholic gel. If need to drink or eat: replace the mask
- When it gets wet, replace it with a new mask
- To remove the mask: remove it from the back (do not touch the front of the mask); immediately throw it in a closed bin or in a bag provided for this purpose; wash your hands with soap and water or with an hydro-alcoholic gel.

Maintenance of the mask : after use, the mask must be isolated either to be thrown away or to be washed. It is recommended to isolate the mask in a disposable plastic bag or in a reusable washing bag. In the case of the reusable bag, mask and bag can be washed together and in this case, take the mask out from the bag.

Washing the mask : Wash in a washing machine at **60°C** with conventional detergent. Then **tumble dry**.

And finally iron at **120°C-130°C**.

This mask can be reused and washed **20 times maximum**.

Storage : Unused masks have to be stored in a clean, dry and dark place.

Wash the mask before first use.



Composition of main material : 2 layers of CVC fabric : 57% cotton /38% polyester / 5% Spandex

Performances : Test protocol described in the French DGA document of March 29, 2020 Category 1 mask «UNS 1» = individual mask for use by professionals in contact with the public. Particle filtration efficiency of 3 µm:> 90%
Breathability: air permeability test> 96 L.m².S-1 for a vacuum of 100 Pa

Textile Masks – UNS Cat.1 – Bulk. MOBILIS [7 Rue Orion, 74650 CHAVANOD] www.mobiliscase.com